

# Flexi-Q PFS

## Fully Disposable Auto-Injector for Pre-Filled Syringes

The Flexi-Q PFS is a fully disposable auto-injector for Pre-Filled Syringes (PFS). This single-use auto-injector features passive needle shielding, allows better patient safety and encourages patient compliance in self-injection. It is the ultimate differentiating vehicle when considering the launch of a new drug or as a lifecycle management tool for an existing drug.



# I3D™

## Injectable Drug Delivery Devices



 Elcam Stopcocks

 Lucomed Medical Products

 Injectech

# Flexi-Q PFS

Fully Disposable Auto-Injector for Pre-Filled Syringes

REV-4 12/2012

## Pharmaceutical Company Advantages

### Differentiation

- Enables using a drug delivery device for drugs in pre-filled syringes
- Flexible delivery volumes of 0.3 - 1.0 mL
- Compatible with low to high drug viscosity
- Reduced skin reactions – injection starts only after full needle penetration
- Delivery of the entire dose – same dead space as with manual technique
- Color coding flexibility and other customization options

### Short regulatory path

- Compatible with standard glass syringes – no need to change drug container


## Patient Advantages

### Ease of use – better patient compliance

- Fully automatic injection process
- Quiet injection reduces patients' anxiety and perceived pain
- Reduces needle phobia – by hiding the needle throughout the injection process


### Patient safety

- Prevents needle-stick injuries – needle is protected throughout the injection process
- Reduced local skin reactions - injects only into the correct depth (SC)
- Clear visual and audible confirmation of the injection progress from beginning to end
- Prevents inadvertent activation - allows activation only after pressing the injector against the patient's skin
- Clear distinction between injector ends – prevents injection in wrong direction

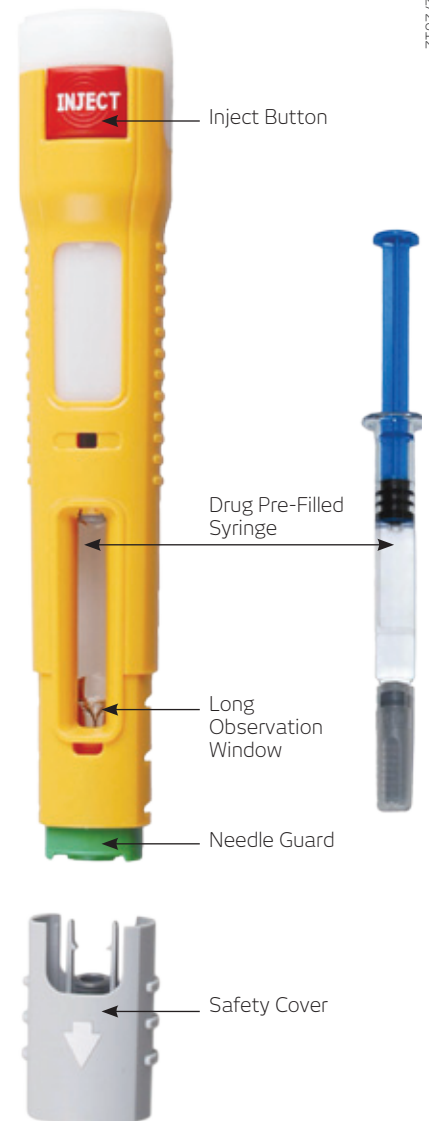

Regulatory status - Device Master File was submitted to the FDA (# MAF-1638)

## See how Easy it is to Use:

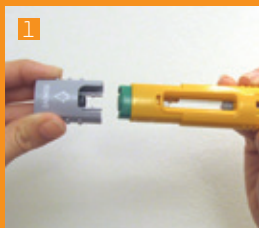

1 Inspect drug integrity through window. Pull out the Safety Cover.



2 Press Injector against skin and press Inject Button. Hold injector still for 10 seconds. You will hear a click at the end of injection. You may also observe the piston reaching the end of the barrel.



3 Remove used auto-injector from skin and discard\* properly.

\* When the auto-injector is removed from the injection site, a Needle Guard automatically covers and locks over the needle for safe disposal of the auto-injector.



[www.elcam-medical.com](http://www.elcam-medical.com)

### Head Office / Manufacturer Elcam Medical

BarAm 1386000,  
Israel  
Tel: 972 4 698 8120/1/2  
Fax: 972 4 698 0777  
menachem@elcam.co.il

### USA Elcam Medical Inc.

2 University Plaza, Suite 620  
Hackensack, NJ 07601  
Tel: +1 201 4571120  
Fax: +1 201 4571125  
info@elcam-medical.com

### Europe Elcam Medical Europe

Via Emilia Romagna 15, 41012  
Carpi (Modena) Italy Tel: +39  
059 697710  
Fax: +39 059 697705  
sales.eu@elcam-medical.com

### China Elcam China Office

Room 3309, Jinhai Mansion  
No. 89 of Anshun Road,  
Sanghai, PRC. Zip: 200051  
Tel: +86 21 62829967  
Fax: +86 21 62829967  
johnliu02@126.com